(ϵ)

Product Name : MANI REAMERS



READ THE INSTRUCTION BEFORE USE

NON STERILE

Before using the MANI REAMERS, please see the instruction manual as indicated below.

arnings

- Only skilled dentists are allowed to use
- 2 Be sure to sterilize this product for each use.
- 3. Do not use this product except for the dental service and treatment Use it in accordance with the intended use

<Contraindications and prohibitions>

Do not use this product for a patient who indicates sensitization and 1. allergic reaction.

<Structures and principals>

- This is the dental file with shank used for enlarging and debriding 1. root canal. It is non-sterile instrument.
- 2 Main materials
 - Cutting part: Stainless steel (including nickel and chromium) 1) Shank : Polybutylene terephthalate (PBT) 2)

<Purpose of use and effect-efficacy>

This is the hand dental rotary instrument for debriding by side cut and enlarging root canal.

<Specifications>

Under the provisions of ISO 3630 performance test, product durability for twisting and bending must exceed specified value of our standard.

<Method of manipulating or method of use>

According to its intended use, choose the most appropriate MANI REAMERS and use it for enlarging and debriding root canal.

<Precautions for use>

- To prevent infection, sterilize the product by autoclave and make sure 1. sterilization is completed before using.
- 2. Choose the most appropriative type for each case and follow the general method.
- 3. Before using, make sure the instruments outside of oral cavity that there are no deformations, scratches and cracks.
- 4. If the head of product is thin, long or large, there are possibilities for breaking or twisting. Because of this, be sure to avoid using unreasonable angle and excessive pressure.
- Wear safety glasses to protect your eyes from damage. Also, wear a 5.dust protective mask to prevent inhaling dust.
- Wear rubber dam etc. to avoid accidental ingestion and falling. 6.
- 7. Do not use this instrument for any purposes except for listed applications above.
- 8. Only for use by dentists.
- This product should be treated as medical waste when disposed. 9.
- 10 Dispose the product if damaged or contaminated.
- After using, wash it with medical cleaning agent and brush, then 11. wash away foreign substances like adherent body fluids and body tissues.
- 12. Set the product to a stand when cleaned by ultrasonic cleaner to avoid deterioration of cutting part.
- Use this product with great care to avoid puncturing fingers because 13.of its possession of sharp-edged part.
- 14 Avoid using unreasonable angle and excessive pressure.**
- This product has possibility that be corrosive if sunk into NaOCl, 15EDTA and etc. for a long time. **

<Storages and duration of use>

- Avoid storing at high temperature, humidity and direct sunlight. 1. Keep liquids away. Store it at room temperature.
- $\mathbf{2}$ Do not damage or make a pinhole to packaging materials.
- 3. The product is subject to be improved without previous notice. Enforce the first-in first-out method for stock management.
- 4. Do not keep under germicidal lamp to avoid deterioration.

<Maintenances and inspections>

- 1. Sterilize this product by autoclave under established method and term for each Method of sterilization:
 - Put this product in a sterilization pack(or foil) and place it on a sterilization tray, or files stand for autoclave sterilization with reference to the following terms Terms for sterilization:
 - term(1) temperature: 121degrees Celsius term(2) temperature: 126degrees Celsius time: 20 minutes or more
 - time: 15 minutes or more
- 2. Do not use autoclave sterilizer which heats more than 200degrees Celsius including drying process.
- When reusing the instrument, wash away foreign substances 3. completely and sterilize.
- Regarding use of medical cleaning agent, follow the instruction 4. manual by its manufacturer strictly.
- Dispose the products if damaged or decreased performance. 5.

<Packagings>

- Minimum packaging unit: 6 pcs./package (#06~#140) 1.
- Assortment: 1 pc. from each size/package 2. (#12~#37, #15~#40, #45~#80, #90~#140)

<Legal manufacturer > MANI, INC.

8-3 KIYOHARA INDUSTRIAL PARK, UTSUNOMIYA, TOCHIGI, 321-3231, JAPAN inquiry and reference: dental department phone: +81-28-667-8592 fax: +81-28-667-8596

<EU representative >

- EC REP Obelis s.a. Bd. Général Wahis 53, 1030 Brussels, Belgium
- * Always keep this document near at hand. In case of loss, please contact dental department of the manufacturer for a conv.
- * The specifications, structures and materials are subject to be changed without previous notices for the demands of improvement.
- * The contents of this instruction manual are subject to be revised without previous notices.
- * * and ** mark indicate revised point.